



COMBAT MEDICAL PRODUCTS



GeM
Government
e Marketplace



"Saving life of a soldier is more important than killing an enemy"

EMCOM 3"&4"

Emcom 3- PVMS No: 29002526, 29002104

Emcom 4- PVMS No: 29002510

All in one complete
emergency dressing



Recommended for:

To provide dressing for
any small injury




The EMCOM 3 & 4 Dressing serves as a versatile solution for covering general wounds on any part of the body. It encompasses all the essential components needed for wound care, including antiseptic, gauze, a cotton pad, a bandage, and tape for securing the dressing.

Additionally, it features a starting adhesive strip that keeps the initial end securely in place, facilitating easy application with one hand or minimal assistance.

Key Features:

- The EMCOM dressing is constructed from a soft, elastic cloth material, ensuring a high level of comfort and unrestricted movement after it's applied.
- The wound pad within the dressing is treated with antiseptic to reduce the risk of infection.
- This wound pad is also protected by a non-stick lining, facilitating effortless and painless removal of the dressing.
- The presence of an adhesive area at the beginning of the dressing aids in the application process and ensures the dressing remains securely in place.
- The EMCOM dressing also features a self-anchoring end, eliminating the need for knots or tapes for additional securing.
- It's delivered in a sterile condition, sealed within vacuum packaging.
- The wound pad can be extended to cover larger wounds as needed.

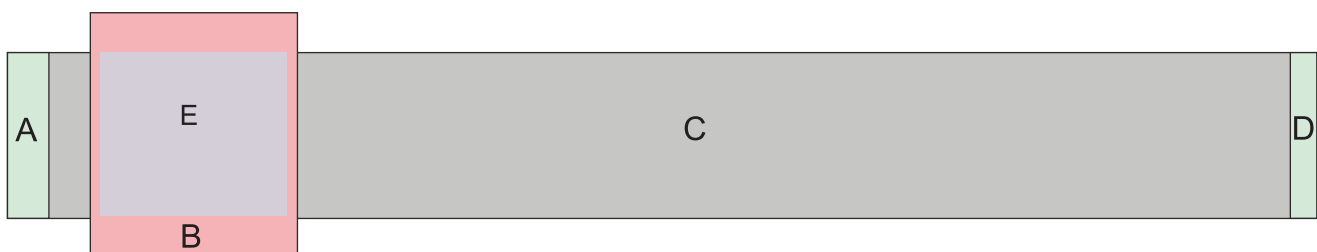

A: Adhesive strip B: Wound pad C: Main Bandage D: End anchor E: Cotton pad

Ordering Information	EMCOM 3"	EMCOM 4"
PVMS No.	29002526, 29002104	29002510
Product Code	Emcom 3	Emcom 4
NSN No.	6510720499289	65107204499292
GeM Catalogue ID	5116877-4337223070-cat	5116877-59077145095-cat
GeM Product ID	7751183-22555956696	7751183-96249987955

EMCOM 3"&4"

All in one complete emergency dressing



Instructions for use



Thigh wound.



Peel open the dressing.



Wrap the dressing around the Limb lightly.



Anchor the end by pressing it on the bandage. End is self anchoring.



Peel off paper liner from the dressing.



Apply adhesive part of dressing before the wound.

PRODUCT PRESENTATION

EMCOM dressings are available in following sizes :

EMCOM3

Length 250 cm (un-stretched)

Width 7.5 cm

Wound cover 12 x 20 cm with antiseptic and non stick lining

Cotton pad 7 x 18 cm

These are supplied sterile.

Double packed in tear open outer and peel open inner packaging

EMCOM4

Length 250 cm (un-stretched)

Width 10 cm

Wound cover 15 x 25 cm with antiseptic and non stick lining

Cotton pad 9 x 23 cm



Made in India by:

G. Surgiwear Limited
Rasoolpur, Jahanganj, Shahjahanpur-242001 (INDIA)
Tel: +91 7518700177 Fax: +91 7518700170
e.mail: info@surgiwear.in
CIN : U24236UP1990PLC012073

Exclusive Distributor:

Medicaid Agencies
UG - 4&5, Shivlok House II
Karampura Commercial Complex
New Delhi - 110015 (INDIA)
Phone: +91-11-41427278, 25920509
Mob. : +91- 9810023232
e.mail : info@medicaidagencies.com

Manufacturing Quality Products for

Neurosurgery	Cardiothoracic
Orthopedics	Laparoscopic
Ophthalmology	Obst. Gynae
Plastic Surgery	Burn Treatment
Urosurgery	Wound Dressing
Andrology	Emergency
General Use	Pressure Dressing

* G Surgiwear Limited retains the authority to make adjustments, modifications, additions, or removals to the products listed in the brochure as deemed necessary, without prior notice. Product specifications are subject to change without prior notice. **EmRescue is registered brand of G Surgiwear Limited.