



COMBAT MEDICAL PRODUCTS



GeM
Government
e Marketplace



"Saving life of a soldier is more important than killing an enemy"

EMCOM CHEST/ABDOMEN

PVMS No: 290033

Chest Dressing



Recommended for:

Chest and Abdominal injuries




EMCOM Chest Dressing serves as a covering for large wounds on the chest or other areas. It combines all the necessary components for wound care into a single unit, eliminating the need for separate items. These components include: antiseptic, gauze, a cotton pad, a bandage, and tape for securing the dressing. Additionally, it features a starting adhesive strip that helps keep the initial end in place, allowing for convenient one-handed or minimal assistance application.

Key Features:

- The EMCOM dressing is crafted from a gentle, elastic fabric that allows unrestricted respiratory movements when applied.
- The wound pad within the dressing contains antiseptic properties, reducing the risk of infection.
- This pad is protected by a non-adhesive lining, ensuring painless and effortless removal of the dressing.
- The presence of an adhesive strip at the beginning of the dressing aids in single hand application and secure stabilization of the dressing, eliminating the need for knots or tapes.
- It features a self-anchoring end, making additional securing measures unnecessary.
- The EMCOM dressing is delivered in a sterile condition, sealed within vacuum-sealed outer packaging.

Specifications:

- Dressing Size: 30 × 200 cm
- Wound Pad Size: 30 × 35 cm

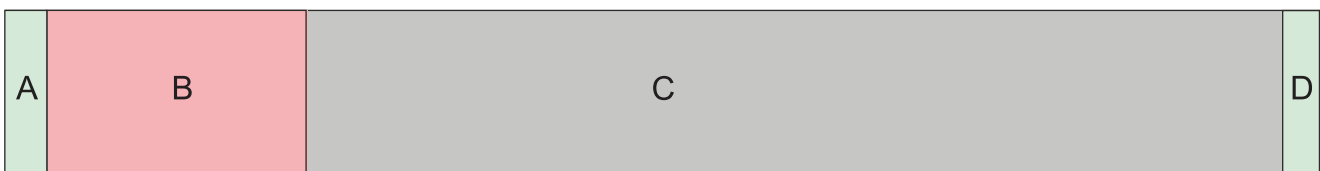

A: Adhesive strip B: Wound pad C: Main Bandage D: End anchor

Ordering information	EMCOM Chest / Abdomen
PVMS No.	290033
Product Code	Emcom Chest
NSN No.	6510720499291
GeM Catalogue ID	5116877-83075277374-cat
GeM Product ID	7751183-38183633439

EMCOM CHEST

Chest Dressing



Instructions for use



1 Penetrating chest wound



2 Peel open the dressing.



5 Wrap the dressing around the Chest lightly.



6 Anchor the end by pressing it on the bandage. End is self anchoring.



3 Peel off paper liner from the dressing.



4 Apply adhesive part of dressing before the wound.

PRODUCT PRESENTATION

EMCOM chest dressing is available in following size.

Length 200 cm (un-stretched)

Width 30 cms

Wound pad: 30 x 35 cms with antiseptic and non stick lining

Emcom chest is supplied double packed, sterilized, vaccum packed with tear notches for ease of opening



Made in India by:

G. Surgiwear Limited
Rasoolpur, Jahanganj, Shahjahanpur-242001 (INDIA)
Tel: +91 7518700177 Fax: +91 7518700170
e.mail: info@surgiwear.in
CIN : U24236UP1990PLC012073

Exclusive Distributor:

Medicaid Agencies
UG - 4&5, Shivlok House II
Karampura Commercial Complex
New Delhi - 110015 (INDIA)
Phone: +91-11-41427278, 25920509
Mob. : +91- 9810023232
e.mail : info@medicaidagencies.com

Manufacturing Quality Products for

Neurosurgery	Cardiothoracic
Orthopedics	Laparoscopic
Ophthalmology	Obst. Gynae
Plastic Surgery	Burn Treatment
Urosurgery	Wound Dressing
Andrology	Emergency
General Use	Pressure Dressing

* G Surgiwear Limited retains the authority to make adjustments, modifications, additions, or removals to the products listed in the brochure as deemed necessary, without prior notice. Product specifications are subject to change without prior notice. **EmRescue is registered brand of G Surgiwear Limited.