











# RAKSHIKA (Isly)

PVMS No: 290074, 290075, 290076

# Emergency Compression bandage

# Instructions for use



The **Rakshika Isly Bandage** features a conventional design tailored to effectively halt blood flow from traumatic bleeding wounds, particularly in critical field conditions. The open "C" shape clip applies ample pressure to the bleeding point while ensuring comprehensive coverage of the injured limb.

The wound pad, designed for comfort and ease of use, is non-adherent, allowing almost painless dressing removal. Furthermore, strategically placed knots within the layers of the bandage prevent it from accidentally unfolding, even if dropped on the ground.


This bandage is equipped with a closure bar at the end, facilitating easy and secure bandage placement. By twisting the closure bar, a tourniquet effect can be generated at the wound site, further aiding in the control of bleeding.

#### **Key features:**

- Rakshika Isly bandage is sterilized and vacuum-packed within durable aluminum packaging.
- This packaging serves as a shield, providing protection against light, ultraviolet radiation, water vapor, oxygen, and microorganisms.
- It is available in three sizes: 4 inches (10 cm), 6 inches (15 cm), and 8 inches (20 cm).
- The 4-inch and 6-inch versions come with an extra wound pad, which proves invaluable in cases involving entry and exit wounds.
- Typically, exit wounds tend to bleed more profusely than entry wounds. In such situations, when both types of wounds are present, the primary wound pad (stitched with the bandage) should be positioned directly over the exit wound, and the additional pad should be adjusted to cover the entry wound.

## Rakshika (Isly) Emergency Compression Bandage with closure bar

Ordering Information	Rakshika (Isly) (4 inch/10cms)	Rakshika (Isly) (6 inch/15cms)	Rakshika (Isly) (8 inch/20 cms)	Rakshika (Isly) (4 inch/10cms) with extra pad	Rakshika (lsly) (6 inch/15cms) with extra pad
PVMS No.	290074	290075	290076	290077	290078
Product Code	RD10CB	RD15CB	RD20CB	RD10CBXP	RD15CBXP
NSN No.	6510720499292	6510720499284			
GeM Catalogue ID	5116877- 99291507551-cat	5116877- 51074394413-cat	5116877- 70149893875-cat	5116877- 99148820909-cat	5116877- 15360335780-Cat
GeM Product ID	7751183- 12281713184	7751183- 24667278518	7751183- 19655588150	7751183- 13261882375	7751183- 65056159107

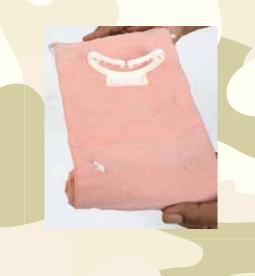





Tear open the pouch and take out the inner pack. Peel open Inner pack & take out the bandage



Remove release liner paper from start of bandage



Place bandage on top of bleeding wound and stick the starting point before the wound



Holding knots provided in between layers of bandage to ensure it would not unfold on its own



Insert bandage cloth into open 'C' shape clip



Pull & wrap bandage in opposite direction forcefully



Keep on wrapping bandage with strongpull



Clip the closure bar onto bandage



Secured bandage

**PRESENTATION** 

Rakshika is availbale in 3 sizes : RD10 (width 10 cm x Length 165 cms) RD15 (width 15 x Length 165 cms) RD20 (width 20 cm x Length 165 cms) Sterilized by Ethylene Oxide.

A C

В

D

1

Ε







### Made in India by:

G. Surgiwear Limited Rasoolpur, Jahanganj, Shahjahanpur-242001 (INDIA) Tel: +91 7518700177 Fax: +91 7518700170

e.mail: info@surgiwear.in CIN: U24236UP1990PLC012073

#### **Exclusive Distributor:**

Medicaid Agencies UG - 4&5, Shivlok House II Karampura Commercial Complex New Delhi - 110015 (INDIA) Phone: +91-11-41427278, 25920509

Mob.: +91- 9810023232

e.mail: info@medicaidagencies.com

### **Manufacturing Quality Products for**

Neurosurgery Cardiothoracic
Orthopedics Laparoscopic
Ophthalmology Obst. Gynae
Plastic Surgery Burn Treatment
Urosurgery Wound Dressing
Andrology Emergency
General Use Pressure Dressing

<sup>\*</sup> G Surgiwear Limited retains the authority to make adjustments, modifications, additions, or removals to the products listed in the brochure as deemed necessary, without prior notice. Product specifications are subject to change without prior notice. \*\*EmRescue is registered brand of G Surgiwear Limited.