



# COMBAT MEDICAL PRODUCTS



GeM  
Government  
e Marketplace



*"Saving life of a soldier is more important than killing an enemy"*

# RAKSHIKA


PVMS No: 290031 | 290032

## Compression bandage for limbs



### Recommended for:

To provide compression on bleeding point of the limbs

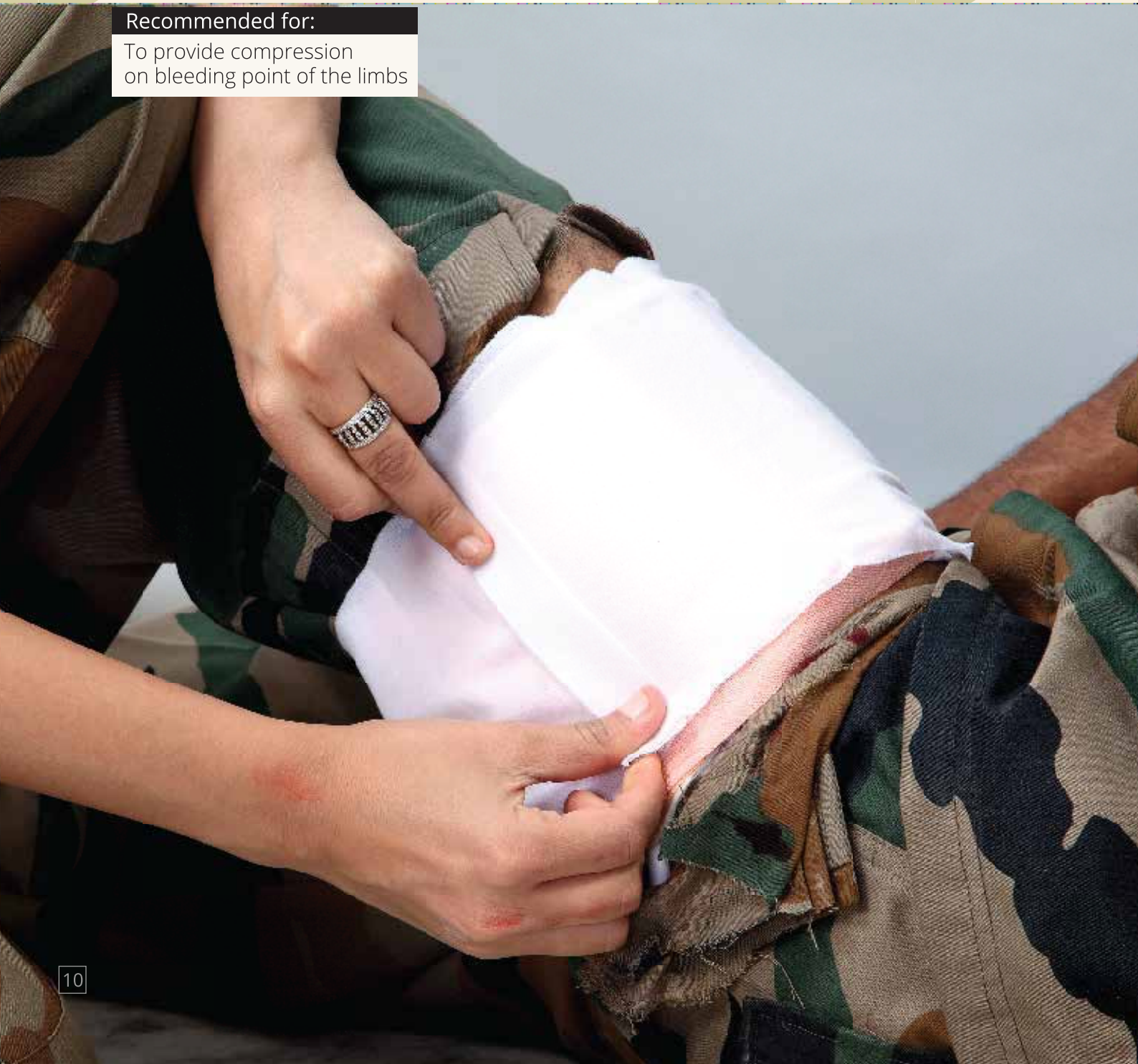

### Rakshika Combat Compression Bandage

represents an innovative and locally designed first-aid bandage tailored for swiftly addressing severe bleeding wounds in emergency scenarios. In situations where every moment is crucial for saving lives, Rakshika bandage stands as the ideal choice for immediate treatment.

While traditional wound dressings require antiseptic, gauze, cotton, and tape to secure it, Rakshika consolidates all these components into one, simplifying the application process. This all-inclusive bandage incorporates various first-aid functionalities, including a primary and secondary dressing, a pressure applicator, and an exceptionally convenient closure method for securing the bandage in place.

The presence of an adhesive strip at the beginning of the dressing aids in single hand application and secure stabilization of the dressing, eliminating the need for knots or tapes. The wound pad includes antiseptic to minimize the risk of infection and is lined with a non-stick material, 'providing almost' and trouble-free dressing removal. Furthermore, strategically positioned knots within the layers of the bandage prevent it from unfolding, even if accidentally dropped on the ground.

### Key features:

- Rakshika bandages, available in both 4-inch and 6-inch sizes, include an additional, unattached wound pad. This extra wound pad proves essential in cases involving both entry and exit wounds, as exit wounds tend to bleed more profusely than entry wounds. In such situations, the primary wound pad, stitched to the bandage, is placed directly over the exit wound, and the supplementary pad is adjusted to cover the entry wound.
- Rakshika bandage is also offered in an 8-inch (20 cm) size, designed for covering larger surface area injuries. While this variant lacks the integrated ridge, pressure can be applied to the wound by cinching the bandage in reverse if necessary.
- Rakshika bandages are exceptionally user-friendly, even to the extent that an injured person can self-apply the bandage with just one hand. They are vacuum-sealed in robust aluminum packaging, which shields them from light, ultraviolet radiation, moisture, oxygen, and microorganisms.
- Rakshika bandage is available in widths of 4 inches, 6 inches, and 8 inches, catering to various wound sizes and types of injuries.



A: Adhesive strip B: Absorbent wound pad C: Rigid ridge D: Main Bandage E: End anchor

Normal systolic blood pressure = 120 mm of Hg  
156 gm/sqcm.

If diameter of limb is 12 cm and width of dressing is 10 cm, you need approx. 9 kg of pull on the dressing to stop bleeding.

In Rakshika without ridge you need 4.5 kg pull to stop bleeding.

With ridge it will be down to 2-3 kg.

Ordering information	Rakshika (4 inch/10 cm)	Rakshika (6 inch/15 cm)	Rakshika (4 inch/10 cm) with extra pad	Rakshika (6 inch/15 cm) with extra pad	Rakshika (8 inch/20cms)
PVMS No.	290032	290031	00000	290078	290076
Product Code	RD10	RD15	RD10XP	RD15XP	RD20
NSN No.	6510720499311	6510720499284			
GeM Catalogue ID	5116877-17033320716-cat	5116877-77648332794-cat		5116877-15360335780-cat	5116877-70149893875-cat
GeM Product ID	7751183-46400604816	7751183-2996740498		7751183-65056159107	7751183-19655588150

# RAKSHIKA

## Compression bandage for limbs

### Instructions for use



1  
Tear open the pouch and take out the inner pack. Peel open Inner pack & take out the bandage



2  
Remove adhesive strip release liner paper at start of bandage.



5  
Press the free flap "3B" against "3A"



6  
Pull the bandage forcefully in opposite direction and wrap it around the limb maintaining strong pull

These are supplied sterile in peel open packaging.  
Vacuum packed with tear notches.



Holding knots have been provided in between layers of bandage so that bandage would not unfold on its own even when it is dropped on ground



Stick the adhesive strip before the wound. The ridge must oppose the bleeding wound.



Keep on wrapping bandage tightly



End part of bandage is cohesive. Press it on the bandage and it will stick on it.

## PRESENTATION

Rakshika is available in 3 sizes : RD 10 (Width 10 cms X Length 250 cms), RD15 (Width 15 cms X Length 250 cms) and RD 20 (Width 20 cms X Length 250 cms)



**Made in India by:**

G. Surgiwear Limited  
Rasoolpur, Jahanganj, Shahjahanpur-242001 (INDIA)  
Tel: +91 7518700177 Fax: +91 7518700170  
e.mail: info@surgiwear.in  
CIN : U24236UP1990PLC012073

**Exclusive Distributor:**

Medicaid Agencies  
UG - 4&5, Shivlok House II  
Karampura Commercial Complex  
New Delhi - 110015 (INDIA)  
Phone: +91-11-41427278, 25920509  
Mob. : +91- 9810023232  
e.mail : info@medicaidagencies.com

**Manufacturing Quality Products for**

Neurosurgery	Cardiothoracic
Orthopedics	Laparoscopic
Ophthalmology	Obst. Gynae
Plastic Surgery	Burn Treatment
Urosurgery	Wound Dressing
Andrology	Emergency
General Use	Pressure Dressing

\* G Surgiwear Limited retains the authority to make adjustments, modifications, additions, or removals to the products listed in the brochure as deemed necessary, without prior notice. Product specifications are subject to change without prior notice. \*\*EmRescue is registered brand of G Surgiwear Limited.