



COMBAT MEDICAL PRODUCTS



GeM
Government
e Marketplace



"Saving life of a soldier is more important than killing an enemy"

THOLOK

PVMS No: 290060, 290062

Chest seal dressing with valve




Recommended for:

Penetrating chest injuries

Also available in twin packing for treatment of both, entry and exit, penetrating injuries.

- Strong adhesive for air tight adhesion to chest

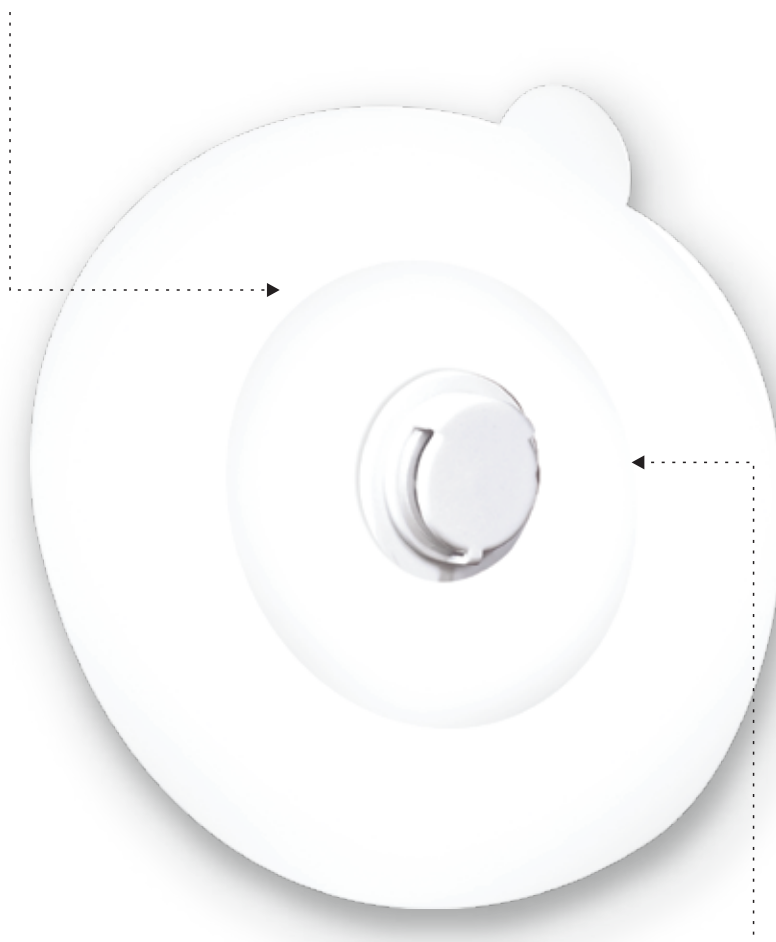

- Ultra low pressure and low profile valve



Tholok Chest Seal is a medical device used for treating open chest wounds, often caused by penetrating trauma like gunshot or stab wounds. It functions by creating an airtight seal over the wound, preventing the entry of air into the chest cavity. This helps alleviate the risk of tension pneumothorax, a potentially life-threatening condition.

Tholok Chest seal is crucial in stabilizing the patient's condition in emergency situations until they can receive proper medical care. Tholok dressing is made of flexible, airtight material with adhesive backing.

Key features:

- Tholok dressing is made up of 15cm diameter strong adhesive coated film.
- Transparent which enables the observation of the wound site.
- The Tholok dressing consists of a low-pressure design facilitating the efficient release of thoracic air.
- The dressing features a low-profile valve that doesn't obstructs other movements.
- The valve is resistant to obstruction or deformation.
- Offered in a dual-pack for addressing both entry and exit wounds resulting from penetrating injuries.
- Supplied sterilized
- Packed with an inner peel-open design
- Each dressing includes a cleaning gauze pad



Ordering Information	Tholok Chest Seal Single pack	Tholok Chest Seal Twin Pack
PVMS No.	290060	290062
Product Code	Tholok V	Tholok V2
NSN No.	6510720499287	
GeM Catalogue ID	5116877-19307311020-cat	5116877-46564968300-cat
GeM Catalogue ID	7751183-72719040257	7751183-36076621765

THOLOK

PVMS No: 290060, 290062

Chest seal dressing with valve

Instructions for use



1 Penetrating chest wound



2 Peel open outer pack and take out Tholok dressing



5 Apply the dressing on the wound taking care to place valve opposite to the wound.




6 Chest seal dressing after application



3

Clean the surrounding area with the sterile gauze pad provided.

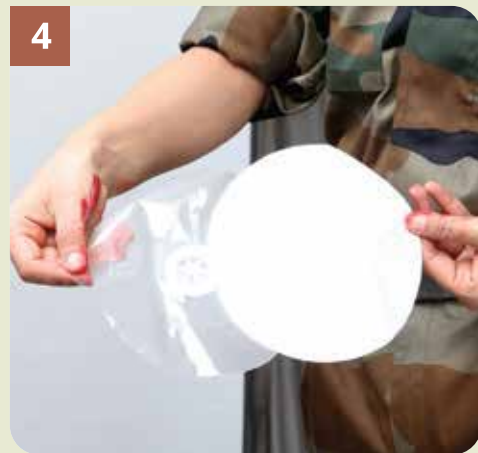


4

Peel off paper liner from the dressing.

PRODUCT PRESENTATION

Dressing with valve
THOLOK-V

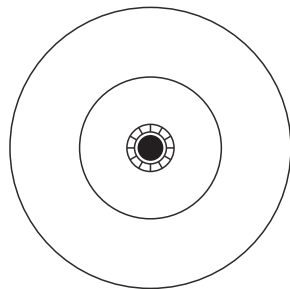

Supplied sterile with gauze pad, packed inside, to clean the wound.

Tholok is available in single as well as twin pack.



Made in India by:

G. Surgiwear Limited
Rasoolpur, Jahanganj, Shahjahanpur-242001 (INDIA)
Tel: +91 7518700177 Fax: +91 7518700170
e.mail: info@surgiwear.in
CIN : U24236UP1990PLC012073

Exclusive Distributor:

Medicaid Agencies
UG - 4&5, Shivlok House II
Karampura Commercial Complex
New Delhi - 110015 (INDIA)
Phone: +91-11-41427278, 25920509
Mob. : +91- 9810023232
e.mail : info@medicaidagencies.com

Manufacturing Quality Products for

Neurosurgery	Cardiothoracic
Orthopedics	Laparoscopic
Ophthalmology	Obst. Gynae
Plastic Surgery	Burn Treatment
Urosurgery	Wound Dressing
Andrology	Emergency
General Use	Pressure Dressing

* G Surgiwear Limited retains the authority to make adjustments, modifications, additions, or removals to the products listed in the brochure as deemed necessary, without prior notice. Product specifications are subject to change without prior notice. **EmRescue is registered brand of G Surgiwear Limited.