



# COMBAT

## MEDICAL PRODUCTS



**GeM**  
Government  
e Marketplace



*"Saving life of a soldier is more important than killing an enemy"*

# EMQUET

PVMS No: 290068, 290069

## Emergency Tourniquet

Recommended for:

To stop bleeding of  
upper and lower limbs.





**Emquet Tourniquet** is a specialized medical device designed for rapid and effective control of severe bleeding in emergency situations, particularly in combat scenarios. It is used to stop the flow of blood from traumatic injuries on limbs, such as gunshot wounds or explosions. This tourniquet is a crucial tool for preventing life-threatening hemorrhage.

**Key features:**

- Quick application
- Durable in challenging environments
- One-handed operation
- Tightening mechanism for adjustable pressure
- Ease of removal once the injured person reaches medical care



Each tourniquet has following components:

1. Main band with combined hook & loop fastener & buckle at one end.
2. An inner twist-able band passing along with stick also known as windlass.
3. Windlass, windlass holding clip with velcro strip attached on top to double secure windlass
4. White patch for mentioning time of application.

Ordering information	EMQUET Emergency Tourniquet
PVMS No.	290068, 290069
Product Code	T01
NSN No.	6515720499285
GeM Catalogue ID	5116877-72337969943-cat
GeM Product ID	7751183-92433394289

# EMQUET

## Emergency Tourniquet

### Instructions for use

- Tear open the pouch and take out the tourniquet.
- Position & apply proximal (near to body) from bleeding point.
- Follow instructions as shown in the pictures:



1 Tear open pouch and take out tourniquet



2 Opened tourniquet



3 Drag tourniquet above bleeding point



6 Twist windlass rod to tighten and create pressure to stop bleeding



7 Push tightened windlass rod into the U clip to hold it in place



8 Secure windlass rod by closing Velcro strap above U clip

**Warning:**  
Emquet Tourniquet must be loosened after one hour. Longer application may cause distal gangrene.



4 Pull Velcro strap to tighten it



5 Wrap the Velcro strap along the circumference of limb



9 Mention time of application at white patch



10 Secured tourniquet

## PRODUCT PRESENTATION

Disinfected before packaging but not sterile.  
Size : Length 95 cms x Width 4 cms



**Made in India by:**

G. Surgiwear Limited  
Rasoolpur, Jahanganj, Shahjahanpur-242001 (INDIA)  
Tel: +91 7518700177 Fax: +91 7518700170  
e.mail: info@surgiwear.in  
CIN : U24236UP1990PLC012073

**Exclusive Distributor:**

Medicaid Agencies  
UG - 4&5, Shivlok House II  
Karampura Commercial Complex  
New Delhi - 110015 (INDIA)  
Phone: +91-11-41427278, 25920509  
Mob. : +91- 9810023232  
e.mail : info@medicaidagencies.com

**Manufacturing Quality Products for**

Neurosurgery	Cardiothoracic
Orthopedics	Laparoscopic
Ophthalmology	Obst. Gynae
Plastic Surgery	Burn Treatment
Urosurgery	Wound Dressing
Andrology	Emergency
General Use	Pressure Dressing

\* G Surgiwear Limited retains the authority to make adjustments, modifications, additions, or removals to the products listed in the brochure as deemed necessary, without prior notice. Product specifications are subject to change without prior notice. \*\*EmRescue is registered brand of G Surgiwear Limited.